

Set-up Instructions

Bed Switch Mount with Light Articulated Arm

- Insert the end of the Light Articulated Arm into the mounting hole in the base plate.
- Rotate the arm to align the flat section with the grub screw then tighten the grub screw.
- Lift the mattress slightly and slide the large aluminium base plate under the side of the mattress adjacent to the pillow - if head height is to be raised and lowered, experiment with positioning of base plate under mattress - usually the best position is over or close to the pivot point where the bed bends when the head is raised.
- Ensure that there is enough clearance between the bed Switch Mount and the bed frame - so the bed height can be adjusted without contacting the mount.
- Adjust each section of the Light Articulated Arm so that its end is positioned near the user's head - or positioned as required by the user.
- Then tighten the three pivot adjustment clamps.

If using a switch:

- Attach switch to end of mounting arm. Run switch lead back along mounting arm and use Velcro straps to secure switch lead to arm as required.

If using a Puff Switch:

- Run Puff Switch tube along mounting arm and position so that the tube protrudes about 5cm to 10cm past end of arm.
- Use Velcro straps to secure tube to arm as required.

