



Cheap Talk 8 Direct, Scan & Jacks

8 scanning possibilities!

Allows the use of one or two switches to "scan" the eight different messages. (See scanning options). This versatile unit also allows direct activation by pressing the squares or 1 to 8 external switches. A light illuminates sequentially. When a window is selected a pre-recorded message will playback. Features scan modes, with a choice of eight different scanning options, including audible scan (see below). The scanning speed is adjustable to accommodate a range of different abilities. Total record time is 40 seconds. Size: 12" x 7½" x 2¼". Requires 4 AA Batteries.

Cheap Talk 8 Direct, Scan & Jacks KA3051

Cheap Talk 4 In-line Direct, Scan & Jacks

Our popular Cheap Talk 4 now scans!

Allows the use of single or dual switch scanning. A light illuminates sequentially above each message square. This versatile unit also allows direct activation by pressing the squares or 1 to 4 external switches. Features scan modes, with a choice of eight different scanning options, including audible scan (see below). The scanning speed is adjustable to accommodate a wide range of abilities. Size: 12" x 6" x 2½". Requires 4 AA Batteries.

Cheap Talk 4 In-line Direct, Scan & Jacks KA3050

8 Different Scan Mode Possibilities!

Advanced computer chip technology allows you to choose one of 8 different scanning modes to meet the needs of the user. In modes 1-6, a signal tone sounds as each light illuminates, with a different tone sounding to indicate the end of a sequence—ideal for visually-impaired users. What Scans? A light under each picture turns on sequentially to indicate the message.

User can also opt to turn on a beep to accompany the light as the unit scans.

Mode 1 Touch & hold external switch to scan. Release to play message.

Mode 2 Touch & release external switch to scan. Touch external switch again to play message.

Mode 3 Touch & hold external switch #1 to scan; release to stop. Touch external switch #2 to play.

Mode 4 Touch external switch #1 to scan; press again to stop. Touch external switch #2 to play.

Mode 5 Manual scan: Touch external switch #1 to advance scan indicator light one square at a time. Touch external switch #2 to play message.

Mode 6 Manual scan: Touch external switch #1 to advance indicator light one square at a time. A pause in manual activation (adjustable from 1-30 seconds) will play the message.

Mode 7 Auditory Cueing: Touch and hold external switch to allow a portion of each message to play. When desired message is reached, release the external switch to hear the entire message. Has one volume control for the entire message.

Mode 8 Auditory Cueing: Touch and release external switch to begin scan. A portion of each message plays until the external switch is touched again, allowing the full message to play. Has one volume control for the entire message.

Icons shown are not included (see page 9)