

Catalogue Number: FLEXSW

FLEXI SWITCH

Most switches used by children and adults with a disability are momentary contact devices. This means the equipment connected to the switch stays on *only* while pressure is maintained on the switch.

The *Flexi Switch* provides latching (press once for on, press again for off), as well as a variable response Timer mode. The variable response period gives a preset 'on' time, even if the user only strikes the switch briefly. This mode is particularly helpful when establishing a cause and effect understanding, and for giving a longer 'reward' period for people who cannot maintain constant pressure on a switch.

The power consumption of the low power *Flexi Switch* is so low, the unit doesn't need an on/off switch!

OPERATING THE FLEXI SWITCH

Plug the lead from the *Flexi Switch* into the toy or equipment to be used.

- If you want a press on, press off action, switch the switch to '**Latch**'.
- For the preset 'on' time mode, switch the switch to '**Timer**' and adjust the knob to give the on time length you want.
- The built in plate switch is very sensitive and should suit most applications.
- An external switch can be plugged in to the 'Switch Input' socket if required, this will disable the plate switch.



BATTERY LIFE

It's hard to say how long your **9Volt battery** will last. Our display unit ran continuously for 14 months and the battery still read its rated voltage. To play it safe, we recommend that if you are not going to use your Flexi Switch for a period of time, remove the battery.

If the unit does not respond to switch activity or behaves erratically, it's probably time for a new battery.

COMMENTS/PROBLEMS?

We are always seeking comments and feedback about our equipment, suggestions for improvement, and ideas for new designs. If you have anything to tell us, we'd love to hear from you.