

*Technical Solutions*

AUSTRALIA PTY. LTD ACN 063 349 502

109 FERNDALE ROAD


SILVAN VIC. AUSTRALIA 3795

PHONE (03) 9737 9000 FAX (03) 9737 9111

## **Information Sheet**

***Fall Monitor2 Floor Sensor Mat Bed Exit Alarm to suit 6mm Mono Nurse Call System***

**Catalogue Number: FMA2 + FMA2-6M**

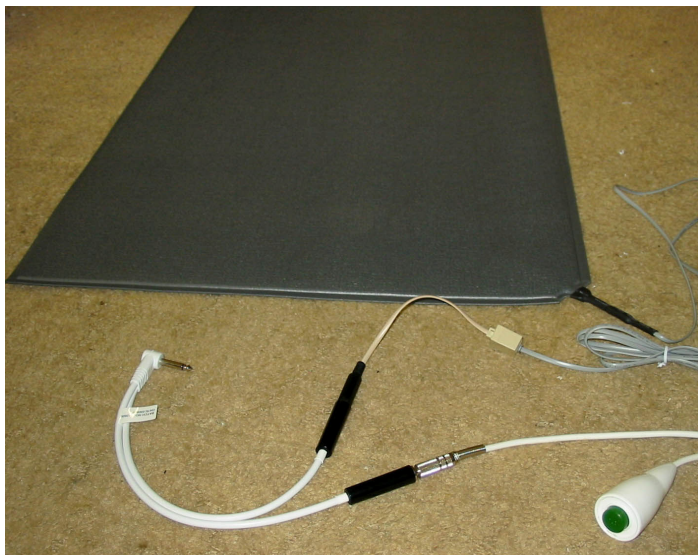

The Sensor Mat is normally placed on the floor beside the bed, in front of a chair or in a door way as required. Standing on the Sensor Mat will trigger an alarm that alerts the nurse or carer.

Using the ***Floor Sensor Adaptor Lead Set*** with 6mm mono plug, (pictured left) the Floor Sensor Mat can be plugged directly into many nurse call systems. The included double adaptor allows the patient's nurse call pendant to plug in as well. Either pressing the call button or standing on the Mat will trigger the nurse call alert.

### **Easy To Set Up:**

1. Place the Sensor Mat on the floor beside the bed – grey side up
  2. If desired, cover the Sensor Mat with a non-slip floor mat – this will both protect and disguise the mat
  3. Unplug the patient's nurse call pendant from the Nurse Call System call point
  4. Plug the white Double Adaptor Lead into the Nurse Call System call point
  5. Plug the patient's nurse call pendant into the white Double Adaptor Lead
  6. Plug the lead from the Sensor Mat into the short Adaptor Lead
  7. Plug the 6mm plug on the Adaptor Lead into the white Double Adaptor Lead
- The Nurse call will be activated when someone stands on the mat
  - Cancel the nurse call system alarm in the normal way

### **Maintenance:**

- Take care of your mat switch. Do not fold, as this will destroy the Sensor. Store flat do not roll. Avoid placing sharp objects on mat
- Take care of your mat switch lead – when moving the bed, do not run over or pull the lead