

Technical Solutions

AUSTRALIA PTY. LTD ACN 063 349 502

109 FERNDALE ROAD

SILVAN VIC. AUSTRALIA 3795

PHONE (03) 9737 9000 FAX (03) 9737 9111

Information Sheet

Fall Monitor2 Floor Sensor Mat Bed Exit Alarm to suit Nurse Call System


Catalogue Number: FMA2-MON2, FMA2

The Sensor Mat is normally placed on the floor beside the bed, in front of a chair or in a door way as required. Standing on the Sensor Mat will trigger an alarm that alerts the nurse or carer.

Using the alarm interface, the Floor Sensor Mat can be plugged directly into many nurse call systems. In some cases we can supply a double adaptor that allows the nurse call pendant to plug in as well. Either pressing the call button or standing on the Mat will trigger the nurse call alert.

Easy To Set Up:

1. Fit 3 x AAA Alkaline Batteries to the Alarm Interface
2. Turn the Alarm Interface on with the small 'ON OFF' switch
3. Place the Sensor Mat on the floor beside the bed – grey side up
4. If desired, cover the Sensor Mat with a non-slip floor mat – this will both protect and disguise the mat
5. Plug the lead from the Sensor Mat into the alarm interface and place the lead under the bed – out of harm's way
6. Plug the short lead from alarm interface into the Nurse Call System call point.
7. Test operation by standing on the mat
8. The Alarm Interface alarm light will start flashing and the unit will start beeping (The beeping can be disabled if required, see below)
9. The Nurse call will be activated
10. Reset the system by pressing the reset button on the Alarm Interface then cancel the nurse call system alarm in the normal way

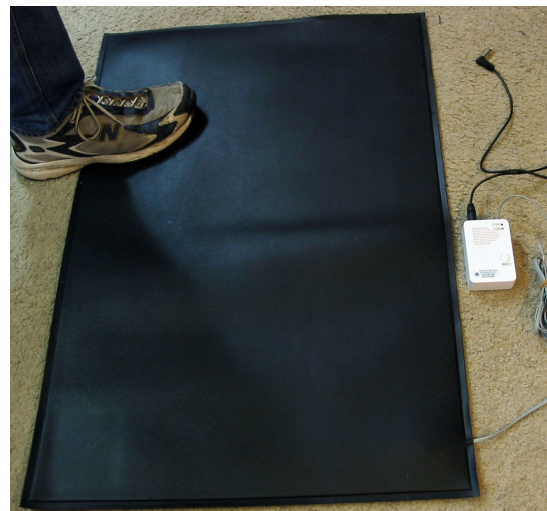

Operation:

11. Press and HOLD the 'Pause' button for 2 seconds till alarm light starts flashing – this will disable the alarm while staff settle patient in the bed
12. Activate monitoring by pressing the 'Pause' button again
- all lights are off when the monitor is active
13. When patient stands on the mat, the alarm will be activated
- reset as per step 9. above

Settings and Maintenance:

- The Alarm Interface has a 'Low battery' warning. Replace when necessary with NEW Alkaline AAA batteries
- **Optional 9VDC Plug pack supply also available from Technical Solutions Australia**
- Take care of your mat switch. Do not fold, as this will destroy the Sensor Store flat do not roll. Avoid placing sharp objects on mat
- The beeper in the Alarm Interface can be disabled by turning on the 'Mute' switch inside the battery cover
- The beeper volume can be adjusted with the 'Volume' switch inside the battery cover