

Technical Solutions

AUSTRALIA PTY. LTD. ABN 14 063 349 502
109 FERNDALE ROAD
SILVAN VIC. AUSTRALIA 3795
PHONE (03) 9737 9000 FAX (03) 9737 9111

Information Sheet

Magic Wand Switch


Catalogue Number: MWPS

Magic Wand Switch

The extremely sensitive touch switch requires absolutely no pressure. It can be operated by touching the built in sensor under the "Technical Solutions" label or by plugging in the sensor wand.

SETUP

- Fit 2 x 'AA' batteries into the battery compartment.
- Plug the sensor wand into the 'Wand' socket.
- Plug the lead from the Magic Wand Switch into the toy or equipment to be operated.
- Position the switch ready for use
- Switch the small 'On/Off/Beep' switch to 'On' or 'Beep'.
In the 'Beep' position the Magic Wand Switch will beep each time it is activated.
- Wait 10 seconds before trying to operate the switch.

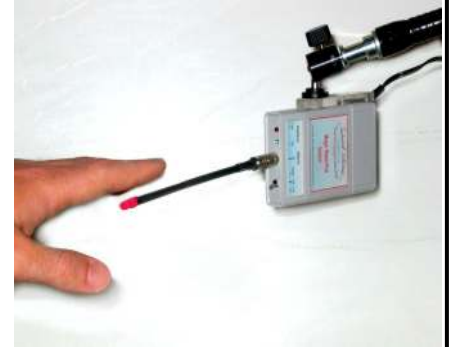

OPERATION

- Moving any part of your body into close proximity with the sensor probe will activate the switch.
- Note that the switch is most sensitive to bare skin.

LIGHTS

- The **green 'OK'** light will come on each time the switch is activated, providing that the batteries are healthy.
- If the **red 'Low'** light will come on each time the switch is activated, this indicates that the batteries are reaching the end of their life.

PROBLEMS

1. The switch does not activate when touched.
 - Remove all parts of your body from close proximity with the sensor, wait 10 seconds and try touching the sensor again.
 - *Energy saver* light globes operating less than ½ Metre from the switch can interfere with its operation – separate the switch and light by 1 metre or more.
 - Remove all parts of your body from close proximity with the sensor, wait 10 seconds and try touching the sensor again.
 - If it still fails to operate, replace the batteries.
2. The switch has activated and will not stop.
 - Turn the switch off and on again or just wait 1 minute and the switch will reset itself.
 - If it still fails to operate, replace the batteries.

Optional Sensors

Magic Big Wand

Our NEW Magic Big Wand is an optional extra add-on for the Magic Wand Switch. It requires absolutely no pressure to operate yet due to its flexibility, it can withstand quite rough treatment from gross body movements. . It can be fitted to any of our mounting systems – mounting arm purchased separately.

Magic Big Wand: MWPS-BW

SETUP


- Screw the Big Wand onto mounting arm of choice
- Plug lead from Big Wand into the ‘Wand’ socket in the Magic Wand Switch
- Use Velcro strap or similar to secure the Magic Wand Switch onto mounting arm
- Turn on the Wand Switch



Magic Headset Switch Sensor

The Magic Headset Switch Sensor is an optional extra add-on for the Magic Wand Switch. It fits round the back of the head, supported by both ears. The headset sensor can be positioned so that it will be operated by small movements of the chin, lips or tongue and requires absolutely no pressure to operate.

MWPS-HS

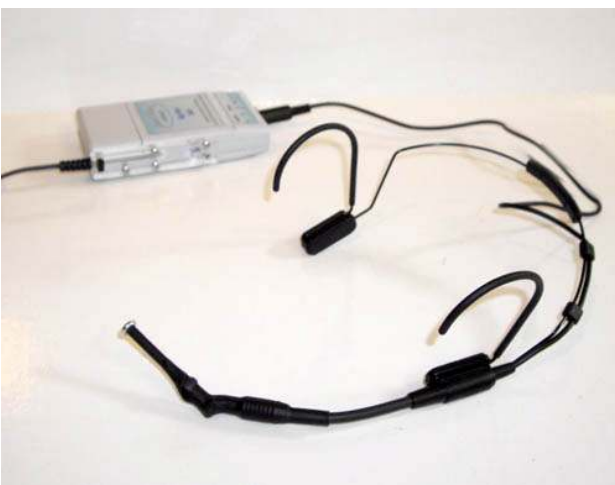

SETUP

- Plug lead from Headset into the ‘Wand’ socket in the Magic Wand Switch
- Secure the Magic Wand Switch onto back of chair so that it does not place strain on the cable or headset
- Adjust flexible arm so that the sensor is close to the chin, lip or tongue.
- Turn on the Wand Switch