

Nurse Call System Connections

Technical Solutions provides a range of equipment for people with special needs - Bed Exit Alarms, Special Accessible Call Buttons, etc. However, as there are no standards for call system wiring, we must wire up each device to suit the particular call system.

The Nurse Call System Adaptor is used to interface our equipment with older style multi-pin call systems.

Setup Procedure

1. Unplug existing call pendant lead from call system
2. Insert the Nurse Call System Adaptor into call system
3. Plug Bed Exit Alarm or Special Accessible Call Button into Adaptor
4. Connect the lead from the plug pack power supply into Adaptor
5. Plug into 240Volt outlet and turn the power on



Operation with Bed Exit Alarm

1. When patient stands on floor mat the **ACTIVE** Led will light and the Nurse Call will be activated.
2. Attend to the patient as soon as possible
3. Cancel the alarm by pressing the **RESET** button

Operation with Special Accessible Call Button

4. When patient presses their Special Accessible Call Button, the **ACTIVE** Led will light and the Nurse Call will be activated
5. Attend to the patient
6. Cancel the alarm by pressing the **RESET** button