



Technical Solutions Australia PTY LTD
ABN 14 063 349 502
109 Ferndale Rd. Silvan VIC Australia 3795
PH (03) 9737 9000
Fax (03) 9737 9111
service@tecsol.com.au
www.tecsol.com.au

Information Sheet

Radio Call Pager *flash/Beep/vibrate*

Cat. No's: **PTX-TE-101**
 PRX-AL903

This unit features a powerful radio transmitter. ***It will operate over distances of up to 500 Metres****. The transmitter can be operated simply by pressing the large blue button or by plugging in an external switch. And ask about our wide range of [Special Access Switches](#) for a switch that suits your needs.

The small convenient **Pager Receiver** is portable and can fit in a pocket or clip onto your belt. The E-Pager Receiver beeps, vibrates and the 6 small lights flash to ensure the attention of the holder. Both the transmitter and receiver are battery powered for convenient portability.

* The range will be dependent on environmental factors.

Set up:

- ◆ Fit 3 x AAA Alkaline Batteries in to the transmitter.
- ◆ Fit 1 x AAA Alkaline Battery in to the receiver.
- ◆ If the user cannot press the BLUE 'call' button on the transmitter, an external switch can be plugged in at the base of the unit.

- ◆ Press the BLUE 'call' button and check that the small RED light on the front of the transmitter flashes once.
- ◆ Also check that the pager (receiver) starts beeping, flashing and vibrating.
- ◆ Reset the Pager Receiver by pressing the reset button on the top of the unit.
- ◆ If using an external switch, check that the pager also operates when this switch is pressed.



Battery Life:

- ◆ The battery life will be dependent on the frequency of operation.
- ◆ The transmitter batteries may last up to a year. For reliable operation, fit new batteries every six months.
- ◆ The receiver battery may last from two to six weeks. The unit will beep with a different tone as an indication that the battery is low.
- ◆ If the operation becomes unreliable, fit new batteries.
- ◆ Always use **NEW Alkaline** Batteries.