

Catalogue Number: RLCB

This unit features a choice of two small radio transmitters. The **Standard Transmitter** will operate over distances of up to 30 metres and can be operated by pressing the "PRESS" button or by plugging in an external switch, as pictured below left - ask about our wide range of **Special Access Switches** for a switch that suits your needs.

The optional **Pendant Transmitter** is operated by pressing the round button on the unit. It will operate over distances of up to 25 metres.



Standard Transmitter



Optional Pendant

The Receiver/Buzzer unit has two modes of operation:

- **Short** - The buzzer will sound only while the call switch is pressed
- **Long** - The buzzer will start when the call switch is pressed and keep buzzing till the carer resets it
- Cancel the alarm by pushing the switch to the **Reset** position

Easy To Set Up:

- The receiver can be moved from room to room to suit the needs of the carer/s
 - Simply plug the receiver power supply into a 240Volt power point
 - Plug the optional Battery Backup into the Receiver, if supplied
- Note: If the red light on the Battery Backup charger is on, or flashing, it is functioning properly



IMPORTANT Maintenance:

Test the transmitter battery regularly with a battery tester (available from Technical Solutions) Replace when necessary, with a NEW Alkaline 9Volt battery or Alkaline 6Volt battery for the pendant. Battery life will vary with frequency of use.

* The range will be dependant on environmental factors.