

**Contents**

1. Jelly Bean Switch
2. Specs Switch
3. Wand Switch
4. Magic Wand Pad switch
5. Buzzer Box (Evaluation Tool)
6. Bed Switch Mount w/JBMount
7. Flexible Switch Mount w/JBMount
8. Boom Arm Stand w/ball swivel & JBmount

Call System Access

Note that a special interface is usually required in order to connect these special switches into a call system. The type of interface will vary – custom made to suit the particular system.

For more information, see our Hospital Call Interface or contact Technical Solutions

**Options**

1. **Jelly Bean Switch** (65mm switch surface area)
  - The Jellybean switch is a very adaptable switch made from durable, shatterproof plastic.
  - Can be used free standing, velcroed to bedside table or other surface, pinned to pillow as a head switch or attached to a switch mount.
2. **Specs Switch** (35mm switch surface area)
  - The Specs switch is a smaller version of the Jellybean at 35mm diameter. Comes with three bases, easy to interchange for flexible mounting
  - Can be used free standing or velcroed to bedside table or other surface. It comes with an elastic Velcro strap that can be used to attach the switch to a bed rail or even to the thumb (see right). The Specs Switch can also be used with a switch mount.
3. **Wand Switch** (8cm wand length)
  - The Wand Switch deals well with 'gross' movements, as the flexible wand will bend but always bounce back to the same position - this switch can handle almost any treatment.
  - Normally used with one of the switch mounts.



#### 4. **Magic Wand Switch** (14cm wand length)

- The Magic Wand-Pad Switch suits a wide range of applications where the user can touch with bare skin, usually the hand, arm or face. The movements can be 'gross' or 'minute' - movements as small as 2mm can be detected and no pressure is required at all!
- **Use as a Wand Switch or as a Pad Switch.**  
The Magic Wand-Pad Switch can be used with the special “No Touch” Wand or unplug the wand sensor and the ‘Technical Solutions’ label on the case becomes the sensitive area giving you a “No Touch” Pad Switch.
- An audible ‘beep’ feedback can be turned on or off as required. It runs for months (depending on frequency of use) on 2 x AA batteries included.
- The switch can be used free standing or used with one of the switch mounts.



#### 5. **Buzzer Box - Evaluation**

- Each switch and switch mounting position can be tested thoroughly with the Buzzer Box to ensure reliable operation before the switch is connected to the call system interface.



#### 6. **Bed Switch mount**

- Used to hold a call switch in position – very useful for patients that have very limited movement.
- Simply slip the metal plate (large for stability) under the mattress, adjust the switch location and tighten the clamps.
- Very useful for adjustable beds as the mount, being held by the mattress will travel with the bed. Keeping the switch positioned in close proximity to the patient.
- Comes with JB adaptor to suit our switches.



#### 7. **Flexible Switch mount**

- The flexible section is 55cm long and a sturdy 18mm in diameter. It comes complete with a strong clamp that anchors onto all kinds of surfaces from bed frames to table edges as well as a swivel ball joint and JB adaptor to suit our switches.



#### 8. **Boom Arm Stand Switch mount**

- Used to hold a call switch in position.
- Usually placed on floor beside the bed with the boom arm positioned over the bed.
- Comes with a swivel ball joint and JB adaptor to suit our switches.

