

## Contents

1. Light Articulated Switch Mount
2. Large Mounting Adaptor
3. Medium Mounting Adaptor
4. Small Mounting Adaptor
5. Big Red Switch
6. Jelly Bean Switch
7. Specs Switch
8. Cuddly Switch
9. Plate Switch
10. Microlight Switch
11. Oval Textured Switch
12. Magic Wand Pad switch
13. Tilt Switch
14. Buzzer Box
15. Flexi Adaptor
16. Battery Tester
17. Screw Driver
18. Velcro Straps



## Setup Options

Any of the switches can be plugged into the Buzzer Box so that the user and therapist can more easily evaluate switch operation. All of these switches can be mounted in various ways and most can be used with the **Light Articulated Switch Mount**.

The **Light Articulated Switch Mount** can be clamped to flat surfaces such as table edges. It can also be clamped to a tubular bed or wheelchair frame after removing the triangular spacer in the clamp jaw. The switches most suitable for use with the mount are as follows.

**Large Target Area Switches (Large Mounting Adaptor):** Big Red or Oval Textured Switch

**Medium Target Area Switch (Medium Mounting Adaptor):** Jelly Bean Switch.

**Small Target Area Switches (Small Mounting Adaptor):** Specs or Microlight Switch.

**Sensitive Switch:** Magic Wand Switch, used with or without the wand attachment.

The Tilt Switch can be strapped horizontally to the head or arm with the Velcro strap. Moving so as to tilt the switch activates it. The Specs Switch has three interchangeable bases that suit various mounting applications – including a ‘belt loop’ style base that can be used to attach the Velcro strap.

All of these switches have a ‘momentary’ action. If you desire a ‘latching’ or ‘timer’ action, use your choice of switch in conjunction with the Flexi Adaptor.