



Technical Solutions Australia Pty Ltd
ABN 14 063 349 502
109 Ferndale Rd. Silvan VIC 3795
PH (03) 9737 9000
Fax (03) 9737 9111
service@tecsol.com.au
www.tecsol.com.au

Information Sheet

TABTIME FLIPTOP

Cat. Number: TIMER-TTFT

TABTIME FLIPTOP DESCRIPTION & INSTRUCTIONS

The Tabtime Fliptop is a pill & medication timer, dispenser & reminder.

Fliptop is a countdown timer designed to operate at set or regular intervals, i.e every 1 1/2 hours, every 2 1/4 -3 1/4 -4- hours and also has a pre dose warning beep 10 minutes & 5 minutes before the actual alarm. You only need to set Fliptop once as it will remember your alarm time for the next dose.

To pause the alarm overnight, press the Start/Stop button after your last dose (The display will be static – no dots flashing when stopped). Begin the sequence again next morning by pressing Start again (The dots will start flashing again)

TO ACTIVATE.

First remove the sliding battery cover on the rear of the case and remove the protective plastic strip. Reinsert the battery and replace the sliding lid. The LCD will now display.

The LCD display shows

The **MONTH** 1-12 on the extreme **LEFT** of the screen

The **DAY** of the month 1-31 on the extreme **RIGHT** of the screen

The **HOUR** next to the month, then the **MINUTE**

TO SET THE TIME

Slide the **TIMER/CLOCK** switch to **CLOCK**

Press the **MW** button and the icon on the extreme left shows the **MONTH** 1-12 press the **HR** button to select the correct month.

Press the **MW** button and the **DAY** icon on the extreme right shows the **DAY** 1-31 press the **HR** button to select the correct day

Press the **MW** button for the **HOUR** and press the **HR** to select the correct **HOUR** Press **MW** for the **MINUTE** and press **HR** to select the correct minute

Press **MW** to confirm the setting and the **:** icon between the **HOUR : MINUTE** will flash to show the **TIME** setting is confirmed

You can reset Fliptop's **TIME & ALARMS** at any time. If at any time you need to reset the programme take out the battery for a few minutes and then start again

TO SET THE ALARM

Slide the **TIME/CLOCK** switch to **TIMER**

Press **HR** button to select how many hours you require to your next dose

Press **MW** to select the minutes.

Press **STAR/STOP/MEMORY** button and your alarm time is now confirmed.

Slide the **TIMER/CLOCK** switch back to **CLOCK**. The alarm is now set

Pressing the **CLEAR** button will, clear the **ALARMS** display

To CANCEL the alarm when beeping, slide the CLOCK switch to TIMER and press STOP. The alarm will now begin the Countdown sequence again.

DISCLAIMER. This product must be regarded as purely an aid to memory. Tabtime Ltd or their agents cannot be held responsible if you forget to take your medication, take the wrong medication, or take the wrong medication at the wrong time